




## Six keys to loading and using the Belmont

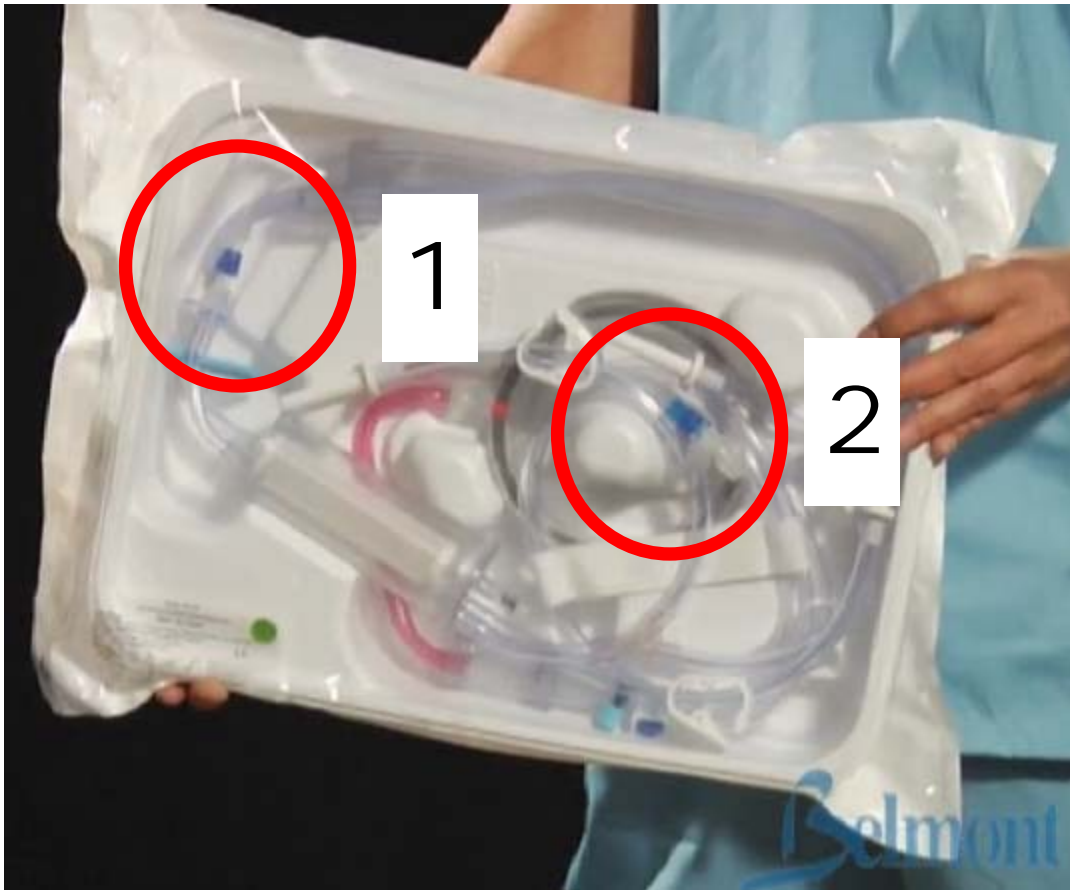

1. Tighten dark blue Luer connector.
2. Tighten dark blue Luer connector.

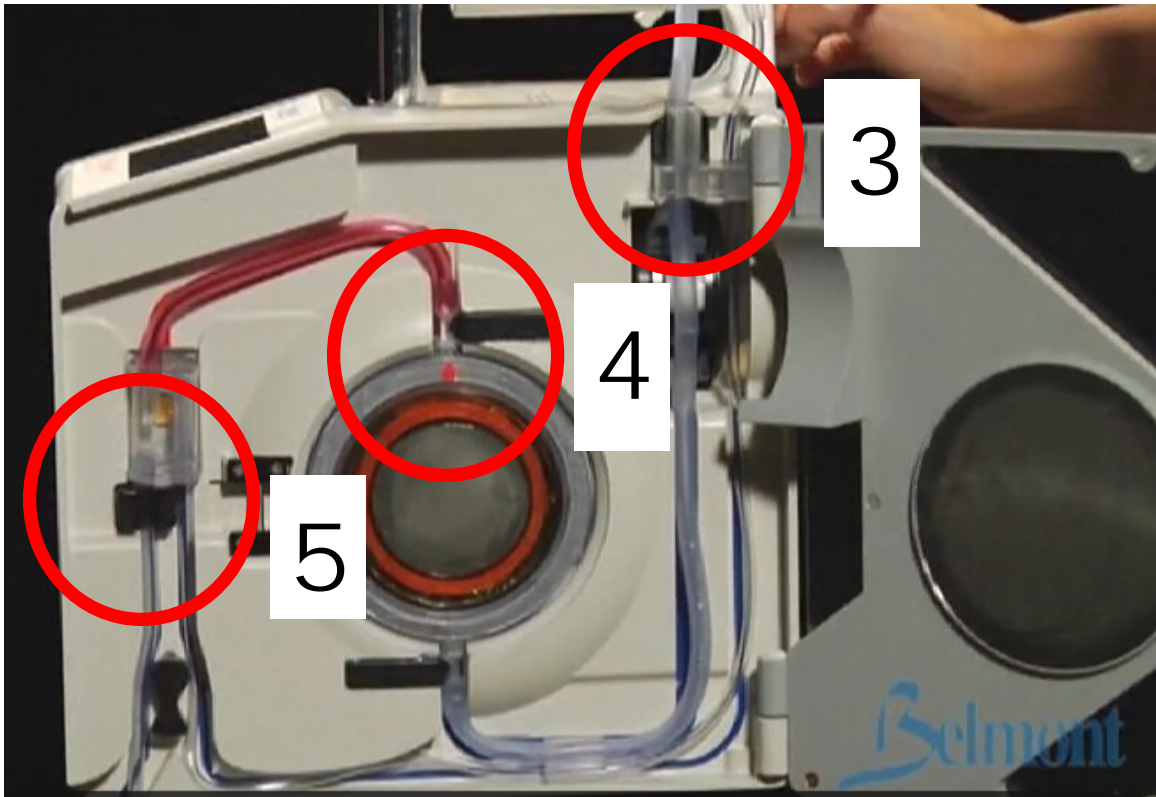

3. Blue arrow on block points into the machine.
4. Red arrow is on front of heater coil and points up.
5. The (larger) air detector tube on the left is seated firmly into the black air detector (very important).

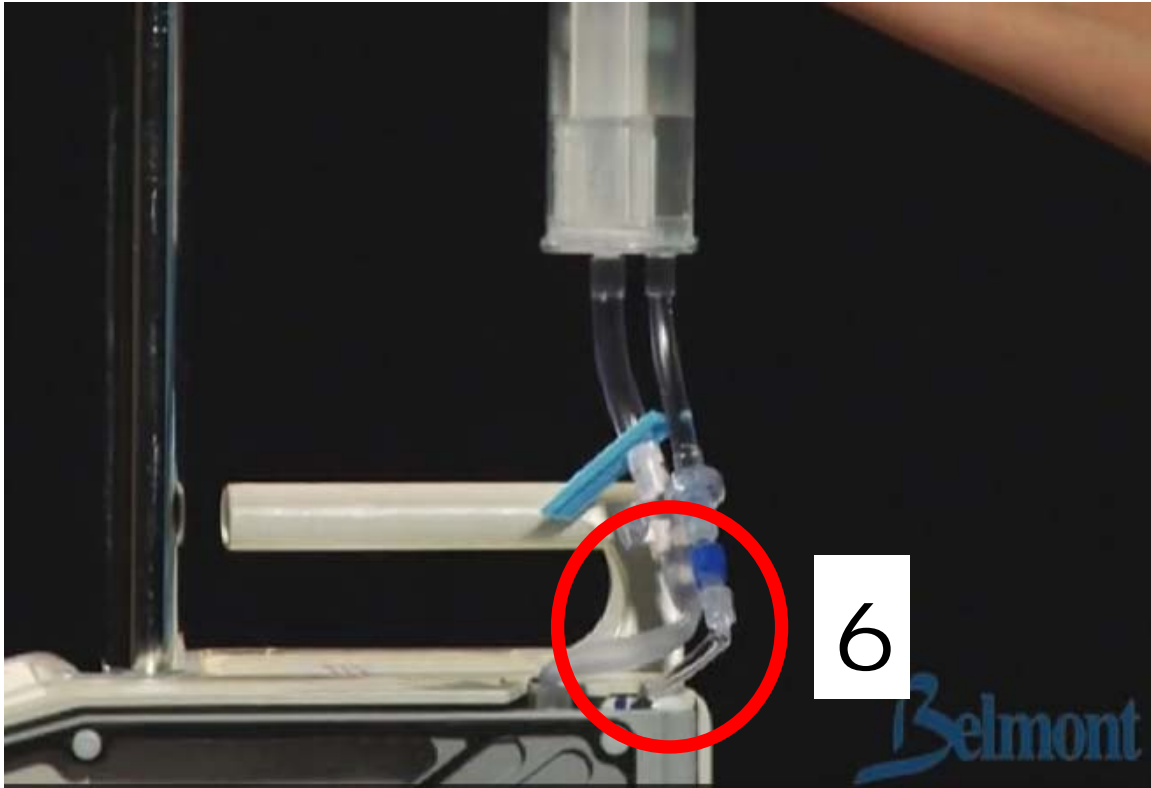

6. Be certain the tubing coming out of the bottom of the reservoir is not stretched or kinked.

If the door closes easily, the Belmont is correctly loaded. If the door doesn't close easily, reload the Belmont.

Now:

1. Turn the Belmont on and follow the instructions on the screen.
2. Spike a solution.
3. Do the "SYSTEM PRIME."
4. PRIME THE PATIENT LINE.
5. Connect the patient line to the patient.
6. Press "INFUSE."